# User manual

MANUEL D'UTILISATION  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
MANUALE DI ISTRUZIONI  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI  
NÁVOD K OBSLUZE

## 385

EN

### Instructions for specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

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Wersja: B, 2024-04

All rights reserved, including translation.

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## 1 Preface

Thank you for your trust in the products of Vermeiren. To support you on the use of this product and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your product.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

### Important note

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures of the product are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

### Information available

On our website, <http://www.vermeiren.com/>, you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

## 2 Your product

1. Expandable tray (if available)
2. Door
3. Drawer
4. Legs or wheels
5. Identification plate

## 3 Before use

### 3.1 Upon delivery

Unpack your product and verify if the delivery is complete. The following items should be included:


- Cupboard (including 4 legs or 2 castors with brake and 2 without brake; with or without expandable tray)
- Instruction manual.

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

### 3.2 Intended use

- This product is a medical device.
- This cabinet, with or without table, is intended to be used near a chair or bed, allowing the user to remain in their wheelchair or bed.
- It is not intended to support or transport goods or persons.
- The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

### 3.3 Safety instructions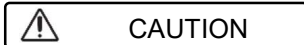

#### Risk of injury and/or damage

- Do not lean against or on the expandable tray, since the cupboard could tip over.
- Do not lift the cupboard by means of the tray.
- Do not place any objects larger than the cupboard on them.
- Do not exceed the bearing capacity of the expandable tray (10 kg).
- Do not convey persons or objects.
- The cupboard must only be set up indoors on level surfaces. When in use, check that all 4 castors / legs press evenly on the floor.
- Repair work may only be undertaken by the authorised dealer.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

Any serious incident [MDR (EU) 2017/745 §2 (65)] that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### 3.4 Shipping and storage

The shipping and storage of the table should happen according to the technical specifications in §7. Make sure that the table is stored dry.


Provide sufficient covering or packaging to protect the table from rust and foreign bodies (e.g. salt water, sea air, sand, dust).

## 4 Assembly

The cupboard is delivered fully assembled and the handrips of the drawer and door are mounted on the inside. You only need to mount the handrips to the outside with a screwdriver.

## 5 Using your table

### 5.1 Operating the brakes (only for cupboard with wheels)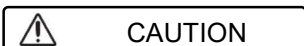

#### Risk of injuries and/or damage


The brakes can wear. Check the working of the brakes before each use.

### **B: Engaging the brakes**

With your foot, press the brake plate downwards till it locks in place. The wheels can now neither roll or turn.


### **A: Disengaging the brakes**

With your foot, press the brake plate upwards. The wheels have now been released and can both roll and swivel.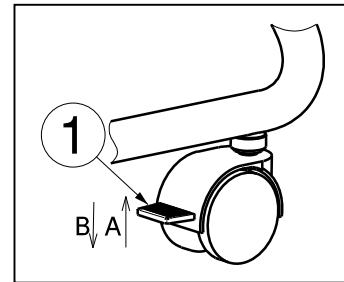

To put the cupboard in a parking position, the two wheels must be locked in place.

### 5.2 Slide out / in expandable tray (if available)

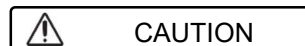The stability of the product is affected when the expandable tray is fold out (danger of tipping over).

#### Slide out

Grasp the expandable tray by the handgrip. Pull the expandable tray backwards and afterwards pull it upwards.

#### Slide in


Lift the tray a little bit upwards and slide the tray in by pushing it down in the cupboard rails.

## 6 Maintenance

### 6.1 Cleaning

Use a soft, damp cloth and lukewarm water with a mild soap to clean the table. Avoid soaking. Do not use abrasive cleaning agents to clean.

### 6.2 Disinfection

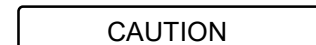#### Risk of damage

Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

Use a soft, damp cloth and lukewarm water with a mild household disinfectant to clean the table.

### 6.3 Inspection / Service / Repairs

The expected service life of your table is 5 years. The lifespan is influenced by use, storage, maintenance, service and cleaning.

Before you resume use of the table, it should be serviced according to the following criteria:

- Completeness
- Castors (stability, tracking, fixing screws)
- Fastenings: well secured (Condition, thread, holding security)
- Cupboard (deformation, stability)
- Tray (closed surfaces, splinters, tears)

Repairs and replacements may only be undertaken by trained persons, and only genuine replacement parts of Vermeiren should be used. Do not hesitate to use the services offered by your specialist dealer. He/she will be glad to assist you regarding any service and repairs.

### 6.4 Reuse

Before each reuse, have the table disinfected, inspected and serviced according to the instructions in §6.

### 6.5 Disposal at end of life

At end of life, you need to dispose of your table according to the local environmental legislation. The best way to do so, is to disassemble the table to facilitate the transport of recyclable parts.

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## Service registration form

This product (name): .....

was inspected (I), serviced (S), repaired (R) or disinfected (D):

|   |   |   |
|---|---|---|
| By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: |
| By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: |
| By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: |
| By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: |
| By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: | By (stamp):<br><br><br><br><br><br><br><br><br><br>Kind of work: I / S / R / D<br>Date: |

Vermeiren GROUP  
Vermeirenplein 1 / 15  
2920 Kalmthout  
BE

website: [www.vermeiren.com](http://www.vermeiren.com)