VERMEIREN

Bobby 24

INSTRUCTION MANUAL
MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

EN

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Manual de instrucciones

Silla de ruedas

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Bimanual wheel-propelled wheelchairs – ISO 9999: 12 22 03

This page is intentionally left blank

Contents

Preface	2
1 Your product	3
1.1 Description	3
1.2 Expected lifespan.....	4
1.3 Declaration of conformity	4
2 Before use	4
2.1 Intended use	4
2.2 General safety instructions	4
2.3 Symbols on the wheelchair	5
2.4 Transport, folding and storage	5
3 Using your wheelchair	9
3.1 Parking brakes	9
3.2 Moving to/from wheelchair	9
3.3 Comfort adjustments.....	10
3.4 Riding.....	10
4 Maintenance	14
4.1 Points of maintenance	14
4.2 Instructions of maintenance	14
4.3 End of use.....	15
5 Technical specifications	15

Preface

Congratulations! You are now owner of a Vermeiren wheelchair!

This wheelchair is made by qualified and committed personnel. It is designed and produced according to high quality standards guarded by Vermeiren.

Thanks for your trust in the products of Vermeiren. To support you on the use of this wheelchair and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your wheelchair.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you on this subject.

Important note

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and/serviced on regular base.

The warranty of this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without being imposed to any obligation to adapt or replace similar products previously delivered.

Pictures are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of following information. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

- This manual
For user and specialist dealer

- Installing instructions (Instructions for assembling and disassembling)
For specialist dealer

- Service manual for wheelchairs
For specialist dealer

- EC declaration of conformity

1 Your product

1.1 Description

This Bobby 24 wheelchair is easy to operate and can be used indoors as well as outdoors. It is foldable which makes it very compact to store. It is available in multiple sizes.

Next picture shows all parts that are relevant for the user. In the operation and maintenance section, these parts are described if applicable.

i Before using your wheelchair, check the technical details and limits of intended use, see chapter 5.

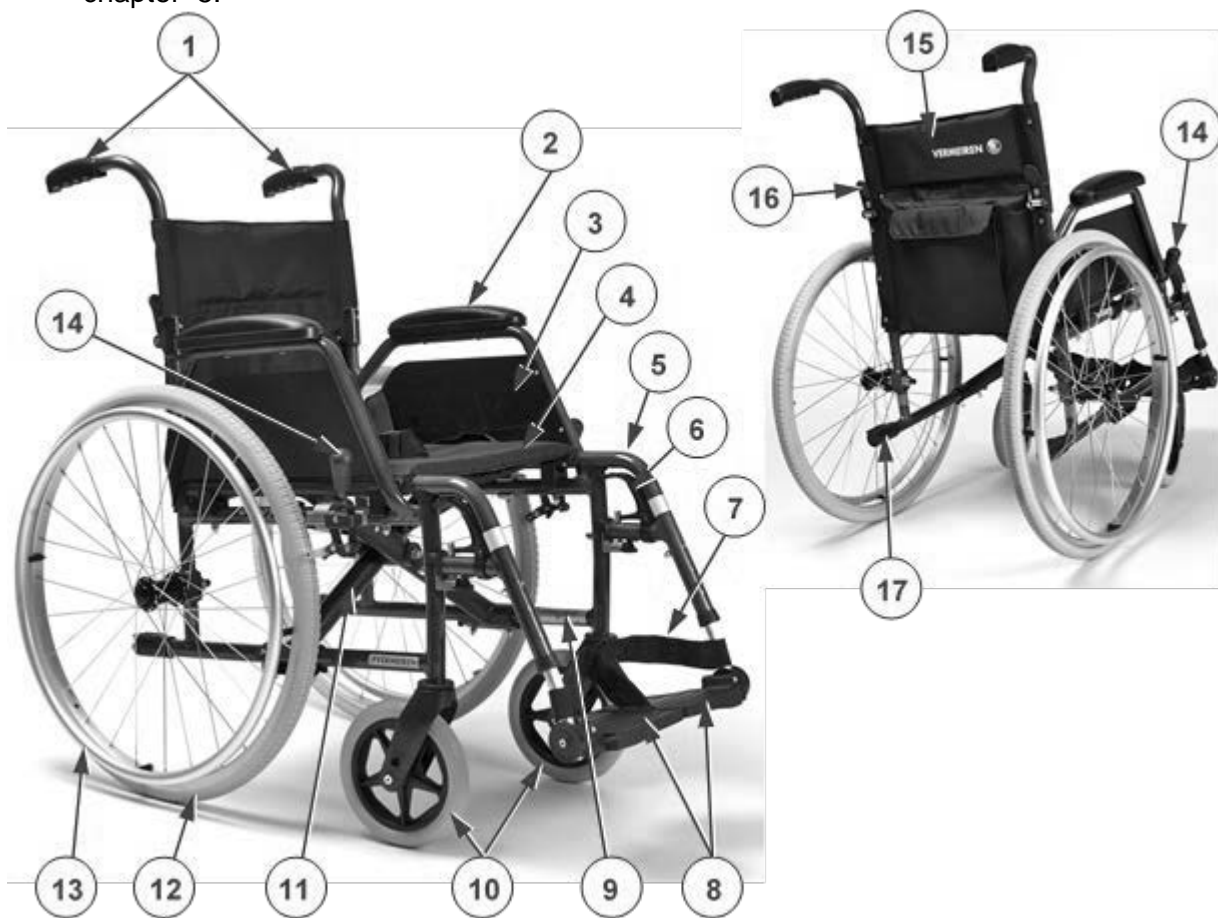
Figure 1 Important parts

- | | |
|--------------------------------------|------------------------------------|
| 1. Handle with hand grip (2x) | 11. Frame cross |
| 2. Armrest (2x) | 12. Driving wheel (rear wheel; 2x) |
| 3. Side plate (2x) | 13. Hand rim (2x) |
| 4. Seat with straps | 14. Brake (2x) |
| 5. Footrest (2x) | 15. Backrest with pocket |
| 6. Footrest release lever (2x) | 16. Backrest folding lever |
| 7. Foot strap (2x) | 17. Tip cap (2x) |
| 8. Footplate (2x) | |
| 9. Location of identification plate | |
| 10. Steering wheel (front wheel; 2x) | |

1.2 Expected lifespan

The average lifespan of your wheelchair is 5 years. Depending on the frequency of use, driving circumstances and maintenance, the lifespan of your wheelchair will increase or decrease.

1.3 Declaration of conformity

This product is certified for CE and is classified as a Medical device, Class I. The CE mark indicates the conformity of the product with standards for protection of health, safety and environment applied on products traded within the European economic area.

A copy of the Declaration of conformity is available via our website: <http://www.vermeiren.com/>.

2 Before use

2.1 Intended use

In this paragraph a brief description of the intended use of your wheelchair is given. Additionally, relevant warnings are added to the instructions in the other paragraphs. In this way we would like to make you aware of the possible misuse that may appear.

- This wheelchair is suitable for indoor and outdoor use.
- This wheelchair is designed and produced solely to transport/transfer one (1) person with a maximum weight of 115kg. It is not designed for transport of goods or objects, nor for any other use than the transfer previously described.
- It is not designed or intended to be used as seat in a motor vehicle.
- The wheelchair can be operated by an attendant or by the user sitting in the wheelchair. You should NOT use this wheelchair without attendant if you suffer from physical or mental impairments that may put you, or other people, in danger when riding the wheelchair. For this reason, consult your doctor first and make sure that your specialist dealer is informed about his/her advice.
- If applicable, only use accessories and spare parts approved by Vermeiren.

Please read all technical details and limits of your wheelchair in §7.

2.2 General safety instructions

 **CAUTION** Risk of injuries and/or damage

Please read and follow the instructions in this manual. Otherwise you may get injured or your wheelchair may get damaged.

Keep next general warnings in mind during use:

- Do not use your wheelchair if you are under influence of alcohol, medicines or other substances that may influence your riding abilities. This counts also for your attendant, operating the wheelchair.

- Be aware that some parts of your wheelchair may get very hot or cold due to ambient temperature, solar radiation or heating devices. Be careful when touching. Wear protective clothing if the weather is cold.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing / care.

2.3 Symbols on the wheelchair

The symbols in following list are applicable for your wheelchair. Symbols can be found in the relevant ISO standard (ISO 7000, ISO 7001 and IEC 417)

Maximum weight of user

Maximum safe slope in ° (degrees).

Not intended to be used as seat in a motor vehicle

Type designation

2.4 Transport, folding and storage

2.4.1 Moving out of the way

Move the wheelchair by using its wheels and roll it to the destination.

If this is not possible and folding is not desired, firmly grasp the frame with both hands and lift it from the floor. Do not use foot or arm rests, or wheels to grasp the wheelchair.

2.4.2 Transport in a vehicle

WARNING

Risk of injuries

Do NOT use your wheelchair as seat in a vehicle, see next symbol.
Never use the same safety belt for passenger and wheelchair.

1. As user, transfer to a seat of the car, see §3.2.
2. Fold the wheelchair to its most compact size according to the instructions in § 2.4.4.
3. Store the wheelchair and its wheels in the luggage compartment. If luggage and passenger compartments are NOT separated or enclosed, fasten the wheelchair frame securely to the vehicle. You can use the available safety belt in the vehicle.

2.4.3 Unfolding

CAUTION

Risk of injuries

- Make sure that your fingers won't get entrapped at the pivot point of the handles.
- Always keep the swing range of the footrest in mind to prevent bystanders to be injured or objects damaged.

[1]

Place the wheelchair with tip caps (17) and front wheels (10) on the ground. Grab armrests (2) and fold the wheelchair open, as much as possible.

[2]

Pull the pelvic belt above the seat. Push seat frame (18) at the left and right side down till the seat is flat and the frame is locked. Check whether the seat is firmly secured.


[3]

Unfold the backrest by pulling handles (1) upwards until the frame tubes are locked in place. Check whether the backrest is firmly secured.

[4]

Take one of the rear wheels (12) and lift the wheelchair on one side. Press and hold button (19) while sliding the wheel axle in the hole of the frame. Release the button to lock the wheel. Repeat this for the other wheel. Check whether both wheels are firmly secured.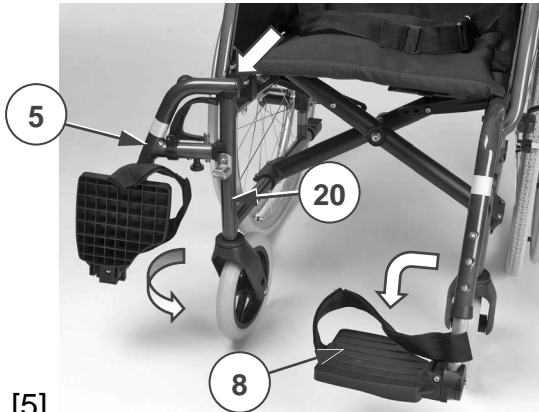

[5]

Hold footrest (5) sideways at the outside of the wheelchair and hang the tube hood of the footrest into frame tube (20).
Swing footrest (5) inwards till it clicks in position.
Swing footplate (8) downwards.
Repeat this for the second footrest.

[6]


Check before use if the footrests and all other parts are securely locked in place.

Your wheelchair is now ready for use.

2.4.4 Folding

 CAUTION Risk of entrapment

Make sure that your fingers are not near the frame cross during folding.

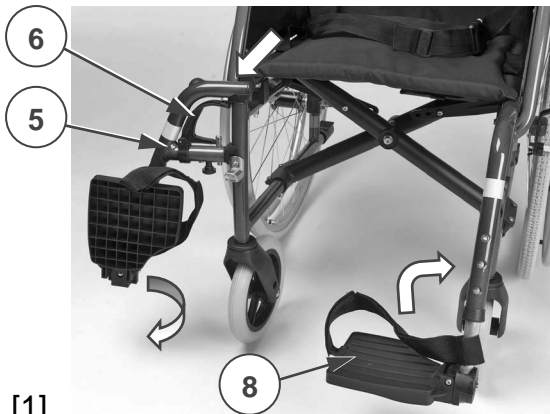
[1]

Turn the footplates (8) upwards.
Pull footrest lever (6) and swing footrest (5) to the outside till it comes loose from the frame tube.

[2]


Remove the wheel by pressing button (19) to unlock.
Take wheel (12) off and place it flat on the ground.
Repeat this for the other wheel.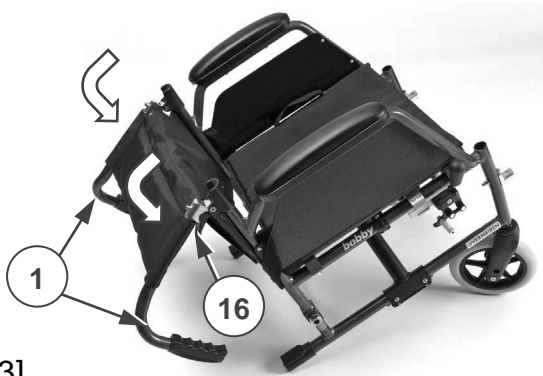

[3]

Press both levers (16) to unlock the backrest tubes.
Bend the upper part of the backrest down by the handles (1) till it is fully folded.

i The backrest can only be folded fully after the wheels have been removed.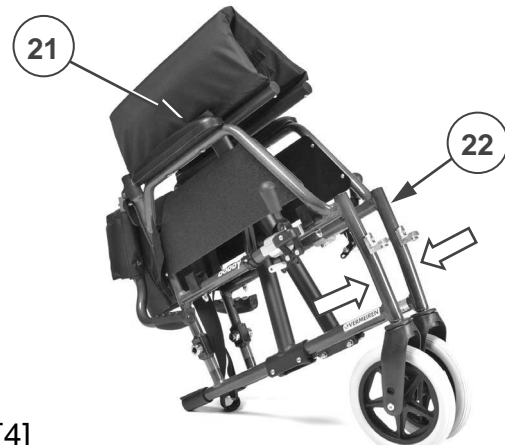

[4]

Fold the wheelchair by pulling both straps (21) of the seat upward. To make it smaller:

- Fold the seat upwards
- Push the left and right frame tube (22) to each other.

The wheelchair is now ready for storage or transport.

2.4.5 Storage

i Make sure that your wheelchair is stored dry to prevent mould to grow and damage the upholstery.
Check the technical details and limits of intended use, see chapter 5.

3 Using your wheelchair

**WARNING**

Risk of injuries

First read previous chapters and inform yourself about the intended use. Do NOT use your wheelchair unless you have read and fully understood all instructions.

In case of doubts or questions, do not hesitate to contact your local specialist dealer, your care provider or technical adviser to help you on this.

3.1 Parking brakes

**CAUTION**

Risk of injuries

- Only operate the brake lever when your wheelchair is stopped.
- An attendant should operate the brake lever if you have impaired mobility.
- Make sure the wheelchair is on a flat horizontal surface before releasing the brakes.
- Never release both brakes simultaneously.
- Brakes may only be adjusted by your specialist dealer.

CAUTION

Risk of damage

Do not use the parking brakes to slow the wheelchair down during movements.

Your wheelchair is equipped with two manually-operated parking brakes. These brakes are intended to prevent movement of the wheelchair when parked.

Releasing brakes [A]:

1. Hold one hand rim (13) tight with one hand.
2. Pull brake lever (14) of the OTHER wheel to the REAR.
3. Repeat this for the second wheel and parking brake.

Applying brakes [B]:

1. Hold one hand rim (13) tight with one hand.
2. Push brake lever (14) of the OTHER wheel to the FRONT till you feel a distinctive lock.
3. Repeat this for the second wheel and parking brake.

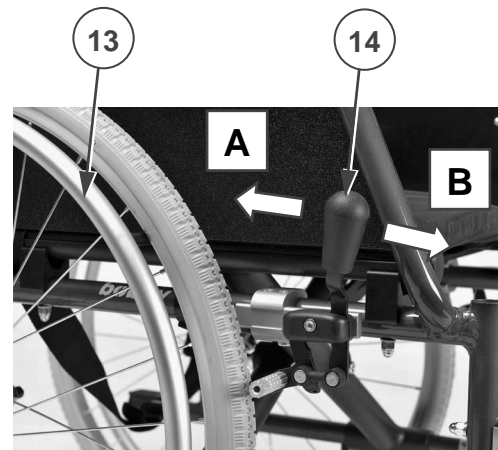
Figure 2 Operating parking brakes

3.2 Moving to/from wheelchair

**CAUTION**

Risk of injuries or damage

- In case you cannot perform the transfer in a safe manner, ask someone to assist you.
- Do not stand on the footplates during transfer into or out of the chair.

Transfer

1. Position the wheelchair as close as possible to you when sitting in your chair, couch, seat or bed.
2. Apply both parking brakes to block the movement, see §3.1.
3. Fold the footplates upwards to prevent standing on them. Or turn the footrests aside to free the passage for transfer.
4. Transfer to/from the wheelchair using the strength of your arms or with the help of lifting equipment.

Sitting in the wheelchair

1. Sit down on the seat with your lower back against the backrest.
2. Turn the footrests back to the front and put your feet on the footplates.
3. Make sure that your upper legs are horizontal and that your feet are in comfortable position; adjust the wheelchair according to the instructions in § 3.3

3.3 Comfort adjustments

 CAUTION Risk of injuries or damage

Do not adjust your wheelchair yourself. Always contact your specialist dealer.

Check next adjustments while sitting in the wheelchair:

- Footrest: the length of the footrest is adapted to the length of your lower leg.
- Footplate: the angle of the footplate is adapted to the position of your feet.
- Parking brakes: both brakes operate smoothly and the wheels don't move when brakes are applied, see §3.1.

If further adjustment is necessary, contact your specialist dealer for support.

3.4 Riding

i This paragraph is a first acquaintance with the operation of your wheelchair. Read the complete manual first before you go for a ride.

 CAUTION Risk of entrapment

- Prevent your fingers from being caught by the wheel spokes.
- Prevent your hands on the hand rims from entrapment when riding through small passages.
- When driving with an attendant, keep your arms away from the wheels and keep your feet on the footplates.

 CAUTION Risk of injuries and damage

- Do NOT drive on slopes, obstacles, steps or kerbs larger than the size described in the technical specification in chapter 5.
- Do not proceed with your wheelchair in traffic. Always stay on the pavement.
- Do not operate the hand rims with wet hands. If you do so, you can lose your grip and lose your balance while coping with slopes or obstacles.
- Pay attention if the road has holes or gaps that may cause entrapment of the wheels.
- Avoid stones and other objects that may block the wheels.
- Always keep the swing range of the footrest in mind to prevent bystanders to be injured or objects damaged.
- Before every usage, make sure that:
 - all footplate and footrest adjustments are firmly secured, see §3.3.

- the safety against unwanted folding/unfolding is functioning properly, see also §2.4.3.
- the parking brakes are working properly.
- the tires are in good condition, see §4.

3.4.1 Riding with an attendant

1. Release the parking brakes or have them released by the attendant, while he/she is holding the wheelchair to prevent movement, see §3.1.
2. The attendant grasps the handgrips.
3. The attendant pushes the wheelchair in the desired direction.
4. After stopping, apply the parking brakes while holding the wheelchair, see §3.1.

3.4.2 Riding on your own

1. Release the parking brakes one by one, see §3.1.
2. Move your hands to the highest position on both hand rims.
3. Lean forward and push/turn the hand rims forward until your arms are straightened.
4. Swing your arms loosely back to the topside of the hand rims and repeat the push movement.
5. To stop: wait till the wheelchair stops and move forward/backward by operating the hand rims.
6. Apply the parking brakes one by one, see §3.1.

3.4.3 Riding on slopes

 CAUTION Risk of injuries

- If the attendant has too little strength to control the wheelchair, stop riding and use the brakes immediately.
- When you stop on a (small) slope, use your brakes, since the wheelchair may start moving unexpectedly.
- Make sure all four wheels touch the ground while riding up or down a slope.
- Take slopes as slow as possible.

1. If available, wear your safety belt.
2. If necessary and if possible, ask an attendant or a bystander to help you.
3. Drive slowly and in a straight line. Lean slightly forward (A/B) when going up the slope. Lean backwards, against the backrest when going down the slope.
4. Never reverse on a slope.

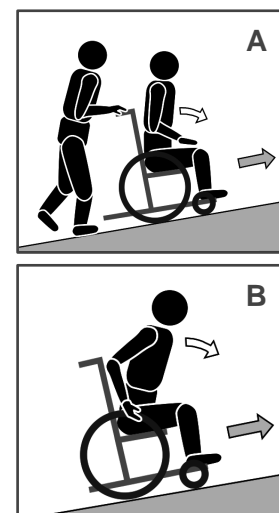
Figure 3 Climbing a slope

3.4.4 Coping with obstacles

CAUTION

Risk of injuries caused by tipping over

- If you have not enough experience driving your wheelchair, ask assistance of an attendant.
- Make sure that the footplates won't touch ground while taking an obstacle.
- Do not use your wheelchair on an escalator.
- If available, wear your safety belt.

Riding without attendant

A practiced user can negotiate small steps or kerbs (up or down) by himself:

1. Move forward approaching the kerb (A).
2. Make sure the footplates won't touch the kerbs.
3. Lean backwards to reduce the pressure on the front wheels (B).
4. Move the wheelchair forward till the front wheels are on the higher/lower part of the kerb.
5. Slowly ride the kerb up or down with the rear wheels. Hold the hand rims firmly to control the movement.

An experienced user can climb higher steps or kerbs by himself:

1. Move forward approaching the kerb to climb.
2. Make sure the footplates won't touch the kerbs.
3. Lean backwards till you balance on the rear wheels and move forward while balancing on 2 wheels (B).
4. When the front wheels are over the kerb, bend forward to get stability (C).
5. Move forward by rolling the rear wheels on the kerb.

An experienced user can descend higher steps or kerbs by himself:

1. Reverses the transfer so that rear wheels approach the kerb first (D).
2. Lean forward to move your centre of gravity to the front.
3. Move close to the kerb.
4. Slowly ride the kerb down backwards. Hold the hand rims firmly to control the movement.


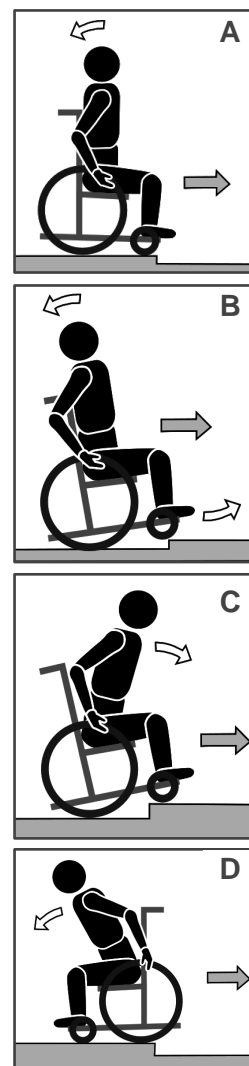
Figure 4 Negotiating kerbs yourself

Riding with an attendant

Small kerbs (up or down) can be taken forward (E / F):

1. The attendant moves the wheelchair forward approaching the kerb. Make sure the footplates won't touch the kerbs.
2. Lean backwards to reduce the pressure on the front wheels.
3. The attendant holds the handles firmly while moving forward. If necessary he uses the tip cap to keep the front wheels lifted till they have passed the kerb rim.
4. The attendant releases the pressure on the handles and tip cap to put the front wheels on the ground.
5. Next, he holds the handles firmly while pushing the wheelchair with the rear wheels up/down the kerb.


Figure 5 Negotiating small kerbs with attendant

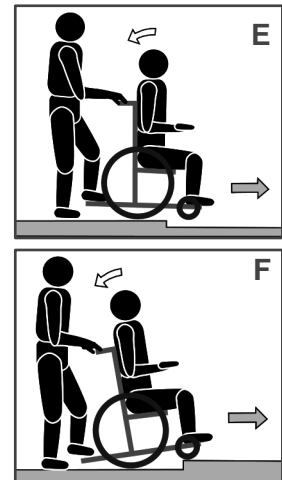Higher kerbs should be taken backwards

1. The attendant reverses the wheelchair so that rear wheels approach the kerb first (G / H).
2. Going down: Lean forward (G) to move your centre of gravity to the front.
Going up: Lean backward (H) to move your centre of gravity to the back.
3. The attendant pulls the wheelchair gently from/on the kerb.

Figure 6 Negotiating high kerbs with attendant

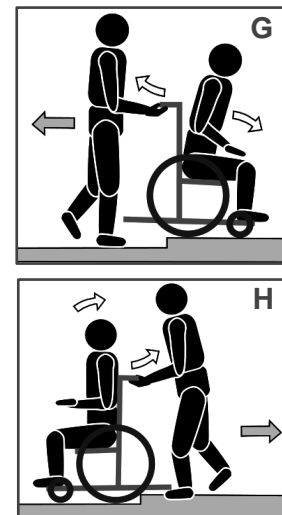3.4.5 Using stairs

 **CAUTION** Risk of injuries caused by tipping over

- Stairs should always be negotiated with the help of two attendants.
- Never try to use stairs that are not appropriate for wheelchairs.

To use the stairs while sitting in your wheelchair, next instructions should be followed:

1. Remove the footrests, see §2.4.4.
2. One attendant tips the wheelchair slightly backwards by the handles.
3. The second attendant grabs the wheelchair at the front of the frame, on both sides of the wheelchair.
4. Stay calm, avoid sudden movements and keep your arms inside the wheelchair.
5. Both attendants lift and pull the wheelchair from step to step while the rear wheels roll over the steps.
6. After using the stairs, mount the footrests back in place, see §2.4.3.

Figure 7 Using stairs

4 Maintenance

- i** Regular care ensures that your wheelchair is preserved in a perfectly functional condition.

4.1 Points of maintenance

CAUTION

Risk of injuries and damage

Repairs and replacements may only be undertaken by trained persons and only genuine replacement parts of Vermeiren should be used.

- i** The last page of this manual contains a registration form for the specialist dealer to record each service.

The service frequency depends on the frequency and intensity of use. Contact your dealer to agree to a common timetable for inspection/maintenance/repair.

Before each use

Inspect next points:

- All parts: Present and undamaged or worn.
- All parts: Clean, see § 4.2.2.
- Wheels, seat, foot rests and footplates: Well secured.
- Condition of wheels/tires, see § 4.2.1;
- Condition of brakes, see § 4.2.1
- Condition of frame: No deformation, instability, weakness or loose connections
- Condition of folding mechanism: Properly working and locking.
- Seat and backrest: No excessive wear (like dented spots, damage or tears).

Contact your specialist dealer for possible repairs or part replacements.

If stored

Make sure that your wheelchair is stored dry to prevent mould to grow and damage the upholstery.

4.2 Instructions of maintenance

4.2.1 Wheels and tires

- i** Proper working of the brakes depends on the state of the tires, that can change due to wear and contamination (water, oil, mud, ...).

- Keep your wheels free of wires, hair, sand and carpet fibres.
- Check the profile of the tyre. If the tread depth is less than 1 mm, the tyres need to be replaced. Contact your specialist dealer for this matter.

4.2.2 Cleaning

Wipe all rigid parts of the wheelchair with a damp cloth (not drenched). If necessary, use a mild soap, suitable for varnishes and synthetics.

The upholstery can be cleaned with lukewarm water and a mild soap. Do not use abrasive cleaning agents to clean.

4.2.3 Disinfection

CAUTION

Risk of damage

Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

4.3 End of use

At end of life, you need to dispose your wheelchair according to the local environmental legislation. The best way to do so, is to disassemble the wheelchair to facilitate the transport of recyclable parts.

5 Technical specifications

The technical details below are only valid for this wheelchair, at standard settings and optimal ambient conditions. Take these details into account during use.

The values are no longer applicable if your wheelchair has been modified, damaged or suffers from severe wear.

Table 1: Technical specifications

Brand	Vermeiren
Address	Vermeirenplein 1/15, B-2920 Kalmthout, Belgium
Type	Wheelchair 
Model	Bobby 24

Description	Dimensions
Max. weight of user	115 kg
Folded width (including wheels)	300 mm
Overall length with footrest	1050 mm
Folded length without footrest	800 mm
Total height	940 mm
Folded height	750 mm
Static stability downhill	10°
Static stability uphill	10°
Static stability sideways	10°

Description	Dimensions	
Obstacle climbing	60 mm	
Seat plane angle	5°	
Effective seat depth	420 mm	
Seat surface height at front edge	500 mm	
Backrest angle	5°	
Backrest height	380 mm	
Angle between seat and leg of footrest	114°	
Distance between arm pad and seat	220 mm	
Front location of armrest structure	315 mm	
Hand-rim diameter	535 mm	
Horizontal location of axle (deflection)	21 mm	
Minimum turning radius	1500 mm	
Diameter Rear wheels	24"	
Tyre pressures	Not applicable	
Diameter steering wheels (front)	200 mm	
Masses of parts that can be removed	Footrest (L/R): 1,60 kg Rear wheel (each): 3,60 kg	
Description	Dimensions Small version	Dimensions Large version
Effective seat width	420 mm	480 mm
Overall width	620 mm	670 mm
Distance between footplate and seat	370 mm	500 mm
Angle between seat and footplate	-1°	19°
Total mass	13,00 kg	13,55 kg
Mass of heaviest part	9,40 kg	9,95 kg
Description	Specification	
Strength tests according to	ISO 7176-8	
Ignitability of upholstery according to	ISO 7176-16	
Use temperature	-25°C ~ +50°C	
Storage temperature	-40°C ~ +65°C	
Storage and use humidity	0% ~ 90%	
We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / 1,5°.		

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Service registration form

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:

Belgium

Vermeiren N.V.

Vermeirenplein 1 / 15

B-2920 Kalmthout

phone: +32(0)3 620 20 20

fax: +32(0)3 666 48 94

website: www.vermeiren.be

e-mail: info@vermeiren.be

France

Vermeiren France S.A.

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59133 Phalempin

phone: +33(0)3 28 55 07 98

fax: +33(0)3 20 90 28 89

website: www.vermeiren.fr

e-mail: info@vermeiren.fr

Italy

Vermeiren Italia

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I-20020 Arese MI

phone: +39 02 99 77 07

fax: +39 02 93 58 56 17

website: www.vermeiren.it

e-mail: info@vermeiren.it

Poland

Vermeiren Polska Sp. z o.o.

ul. Łączna 1

PL-55-100 Trzebnica

phone: +48(0)71 387 42 00

fax: +48(0)71 387 05 74

website: www.vermeiren.pl

e-mail: info@vermeiren.pl

Czech Republic

Vermeiren ČR S.R.O.

Nádražní 132

702 00 Ostrava 1

phone: +420 596 133 923

fax: +420 596 121 976

website: www.vermeiren.cz

e-mail: info@vermeiren.cz

Germany

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D-40472 Düsseldorf

phone: +49(0)211 94 27 90

fax: +49(0)211 65 36 00

website: www.vermeiren.de

e-mail: info@vermeiren.de

Austria

Vermeiren Austria GmbH

Schärdinger Strasse 4

A-4061 Pasching

phone: +43(0)7229 64900

fax: +43(0)7229 64900-90

website: www.vermeiren.at

e-mail: info@vermeiren.at

Switzerland

Vermeiren Suisse S.A.

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3645 Gwatt (Thun)

phone: +41(0)33 335 14 75

fax: +41(0)33 335 14 67

website: www.vermeiren.ch

e-mail: info@vermeiren.ch

Spain / Portugal

Vermeiren Iberica, S.L.

Carratera de Cartellà, Km 0,5

Sant Gregori Parc Industrial Edifici

A 17150 Sant Gregori (Girona)

phone: +34 972 42 84 33

fax: +34 972 40 50 54

website: www.vermeiren.es

e-mail: info@vermeiren.es

Manufacturer

Vermeiren GROUP

Vermeirenplein 1 / 15

B-2920 Kalmthout

phone: +32(0)3 620 20 20

fax: +32(0)3 666 48 94

website: www.vermeiren.com

e-mail: info@vermeiren.com

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All rights reserved, including translation.