VERMEIREN

Lola - Lory

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

EN

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EN Contents

1. Technical details 2
2. Components 2
3. Explanation of symbols 2
4. Check after delivery 2
5. Intended use 3
6. Before use 3
7. Assembly shower chair 3
8. Seat height adjustment 3
9. Carrying the shower chair 3
10. Safety instructions 3
11. Maintenance, inspection, cleaning 3

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INSTRUCTION MANUAL
Shower chairs "Lola, Lory"

1. Technical details

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

Make	Vermeiren
Address	Vermeirenplein 1/15, B-2920 Kalmthout
Type	Shower chair
Model	Lola
Width	450 mm
Length	420 mm
Height	690 mm - 790 mm
Seat height	420 mm - 520 mm
Seat width	450 mm
Seat depth	340 mm
Backrest height	270 mm
Weight total	2,85 kg
Patient weight	max. 130 kg
Model	Lory
Width	450 mm
Length	420 mm
Height (Seat height)	420 mm - 520 mm
Seat width	450 mm
Seat depth	340 mm
Weight total	2,10 kg
Patient weight	max. 130 kg
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg	

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.



Before using the shower chair, read the instruction manual carefully: it will familiarise you with this product.

If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

Visually impaired people can contact the dealer for the instructions for use.

2. Components

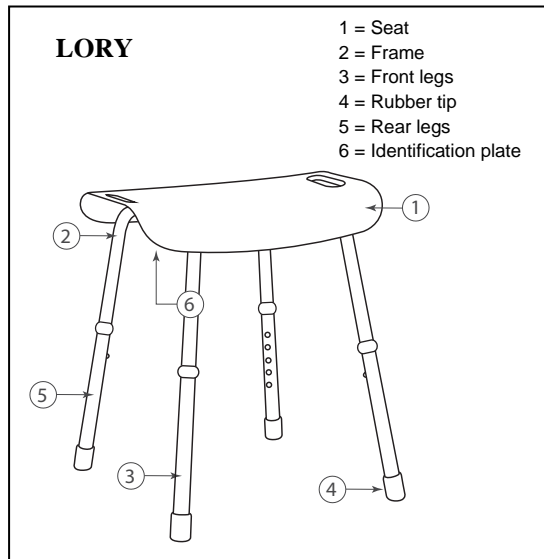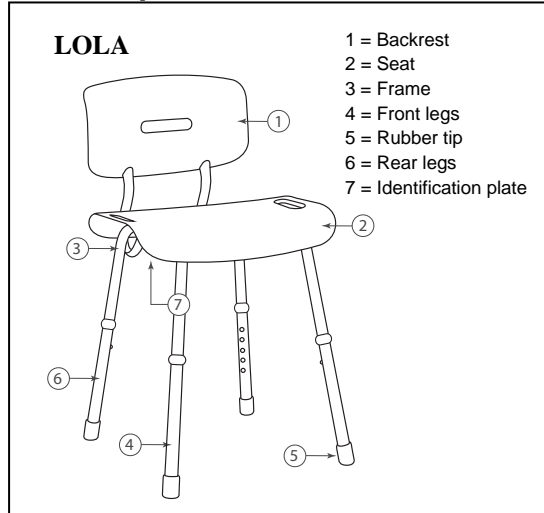3. Explanation of symbols

Maximum mass

CE conformity

Type designation

4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- Seat
- Backrest (only for Lola)
- 4 Height adjustment legs (on the bottom) + rubber tips
- 5 screws + washers
- 2 pair of legs (on the top)
- Manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems

- contact your supplier

5. Intended use

The shower chair shall be used by 1 person.
The shower chair shall be used indoor, in the shower, bath

All our shower chairs are ideal for usage in the shower, bathtub, bathroom or other humid environments. The specially designed rubbers at the end of each leg will create a safe stability for the chair and avoid the sliding of the chair.

The construction of the chair allows full use by elderly people or during revalidation in home environment.

The chair shall not be used as toy for children. The chair should only be used on horizontal, non smooth surfaces.

The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

6. Before use

- Remove all packaging before using the shower chair for the first time.
- Assembly of shower chair.
- Adjust the seat in the desired height.

7. Assembly shower chair


Refer to the attached assembly drawings.

8. Seat height adjustment

⚠ CAUTION: Risk of falling - Secure the seat firmly before using the shower chair. Check that the spring pin ① is firmly fixated in a hole.

⚠ CAUTION: Risk of falling - Do not exceed the maximum seat height adjustment, your shower chair will not be stable anymore.

⚠ CAUTION: Risk of tipping over - Place the four legs at the same height.

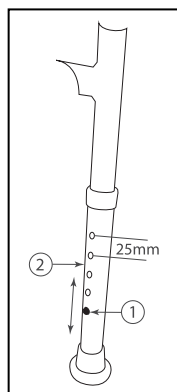Before using the shower chair you must adjust the seat in the correct height.

The seat can be adjusted in height by following steps:

- Press on the spring pin ①.
- Move the leg ② until the desired height (5 different positions: steps from 25 mm).
- Check that the spring pin ① is clicked back and firmly fixated in a hole.
- Adjust the other legs with the same instructions.
- Check that all legs are placed in the same position.

9. Carrying the shower chair

The best way to carry the shower chair Lola: grasp the handgrip of the backrest.

The best way to carry the shower chair Lory: grasp the handgrips of the seat with two hands.

10. Safety instructions

- **Do not exceed the maximum user weight. The maximum user weight for your chair is 120 kg.**
- Use the shower chair on horizontal, non-smooth surfaces, all four legs must have contact with the ground.
- Keep it away from children. The shower chair shall not be used as a toy.
- When it is used by, on or near children, invalids or disabled persons close supervision is necessary.
- Risk of burns – Be careful when using in extreme hot or cold environments for a sufficient amount of time and when touching.
- Do not use the shower chair when it is not working properly.
- Check that the shower chair is firmly assembled before using the chair.
- Do not lift the shower chair when the user is seated on the chair - Risk of injury or damage.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

11. Maintenance, inspection, cleaning

For the maintenance manual of the shower chair refer to the Vermeiren website: www.vermeiren.com.

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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VERMEIREN GROUP
Vermeirenplein 1/15
B-2920 Kalmarhout
BELGIUM

Notes

A series of horizontal dotted lines for writing notes.

Notes

A series of horizontal dotted lines for writing notes.

Belgium

N.V. Vermeiren N.V.

Vermeirenplein 1 / 15
B-2920 Kalmthout
Fax: +32(0)3 666 48 94
website: www.vermeiren.be
e-mail: info@vermeiren.be

France

Vermeiren France S.A.

Z. I., 5, Rue d'Ennevelin
F-59710 Avelin
Tel: +33(0)3 28 55 07 98
Fax: +33(0)3 20 90 28 89
website: www.vermeiren.fr
e-mail: info@vermeiren.fr

Italy

Vermeiren Italia

Viale delle Industrie 5
I-20020 Arese MI
Tel: +39 02 99 77 07
Fax: +39 02 93 58 56 17
website: www.vermeiren.it
e-mail: info@vermeiren.it

Poland

Vermeiren Polska Sp. z o.o

ul. Łączna 1
PL-55-100 Trzebnica
Tel: +48(0)71 387 42 00
Fax: +48(0)71 387 05 74
website: www.vermeiren.pl
e-mail: info@vermeiren.pl

Czech Republic

Vermeiren ČR S.R.O.

Nádražní 132
702 00 Ostrava 1
Tel: +420 596 133 923
Fax: +420 596 121 976
website: www.vermeiren.cz
e-mail: info@vermeiren.cz

Germany

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Wahlerstraße 12 a
D-40472 Düsseldorf
Tel: +49(0)211 94 27 90
Fax: +49(0)211 65 36 00
website: www.vermeiren.de
e-mail: info@vermeiren.de

Austria

Vermeiren Austria GmbH

Schärddinger Strasse 4
A-4061 Pasching
Tel: +43(0)7229 64900
Fax: +43(0)7229 64900-90
website: www.vermeiren.at
e-mail: info@vermeiren.at

Switzerland

Vermeiren Suisse S.A.

Eisenbahnstrasse 62
3645 Gwatt (Thun)
Tel: +41(0)33 335 14 75
Fax: +41(0)33 335 14 67
website: www.vermeiren.ch
e-mail: info@vermeiren.ch

Spain / Portugal

Vermeiren Iberica, S.L.

Carratera de Cartellà, Km 0,5
Sant Gregori Parc Industrial Edifici A
17150 Sant Gregori (Girona)
Tel: +34 972 42 84 33
Fax: +34 972 40 50 54
website: www.vermeiren.es
e-mail: info@vermeiren.es

Manufacturer

Vermeiren GROUP

Vermeirenplein 1 / 15
B-2920 Kalmthout
Tel: +32(0)3 620 20 20
Fax: +32(0)3 666 48 94
website: www.vermeiren.be
e-mail: info@vermeiren.be

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All rights reserved, including translation.