## Shower benches

All our shower benches are ideal for usage in the shower, bathtub, bathroom or other humid environments. The specially designed rubbers at the end of each leg will create a safe stability for the bench and avoid the sliding of the bench.

All models have leg adjustments. You can easily adjust the length of the legs by pressing the snap buttons and sliding the tube in- or outwards. Make sure the snap button securely locks into the hole. Ensure that all four legs are of the same height before using your shower bench.

 Ask assistance if you are uncertain on the usage of your product.

 Users weight: 150 kg

The shower benches can be cleaned with normal household cleaning products. Avoid using aggressive cleaning products which can cause discoloration.


### Dimensions Jill

width: 51 cm  
length: 42 cm  
seat height: 35 - 53 cm  
seat width: 51 cm  
leg adjustment: 15 cm  
weight: 2 kg


### Dimensions Jinny

width: 35 cm  
length: 35 cm  
seat height: 33 - 51 cm  
seat width: 32 cm  
leg adjustment: 15 cm  
weight: 1,5 kg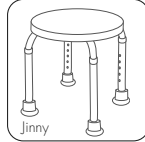


### Dimensions Lilly

width: 51 cm  
length: 47 cm  
height: 77 - 87 cm  
height backrest: 38 cm  
seat width: 51 cm  
seat height: 36 - 46 cm  
leg adjustment: 15 cm  
weight: 3 kg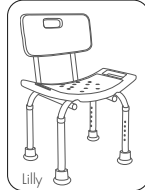

### Dimensions Melbourne

width: 48 cm  
length: 56 cm  
height: 72 - 82 cm  
height backrest: 37 cm  
seat width: 41 cm  
seat height: 34 - 44 cm  
leg adjustment: 15 cm  
weight: 2,8 kg

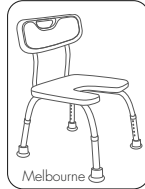### Dimensions Sidney

width: 50 cm  
length: 52 cm  
seat height: 34 - 44 cm  
seat width: 41 cm  
leg adjustment: 15 cm  
weight: 2 kg

Subject to technical alterations. © 2016

**VERMEIREN**

Opening up new horizons in mobility

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Vermeiren Nederland B.V.