# User manual

MANUEL D'UTILISATION  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
MANUALE DI ISTRUZIONI  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI  
NÁVOD K OBSLUZE

## Kate

EN

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Verze: A, 2023-06

All rights reserved, including translation.

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## Shower bench Kate

### 1 Upon delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- 2 boxes with screws and washers
- 8 metal frame parts
- 4 plastic frame parts
- Manual + Assembly guide

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

### 2 Assembling the shower bench

**CAUTION**

Risk of injuries

- Make sure that all screws have been tightened securely before using your shower bench.

To assemble Kate:

1. Set the 4 legs to the same height with the snap button.
2. Secure the handle attachments into place.
3. Assemble the connection pieces according to their labelling (side 1, side 2).
4. Install the 3 seat plates.
5. Mount the handle with the snap buttons.
6. Assemble the backrest.
7. Mount the backrest with the snap buttons.

The back support is reversible to accommodate left or right, depending on which side you want the armrest.

If you have any doubts, do not hesitate to contact a professional to assemble Kate for you.

### 3 Using the shower bench

**CAUTION**

Risk of injuries

- Be careful when using in extreme hot or cold environments (sunshine, extreme cold, saunas, etc.).
- If you notice any defects, refrain from using Kate.
- Make sure your shower bench is always on an even and smooth surface before usage.
- Make sure all snap buttons are tightly into place and that all four legs are of the same height.

The maximum user weight for the shower bench is 150 kg.

### 3.1 Seat height

To adjust the length of the legs:

1. Press the snap button and slide the tube until the desired length.
2. Make sure the snap button is locked into position before using your bench again.
3. Repeat for the remaining 3 legs.

### 4 Maintenance

Kate can be cleaned and disinfected with normal household products. Avoid using abrasive cleaning products that can cause discoloration.

The high quality of the surface layer guarantees optimal protection against corrosion. If the outer coat is damaged by scratching or in some other way, get your specialist dealer to repair the affected surface.

#### 4.1 Expected lifespan

The shower bench is designed to have an average lifespan of 5 years.

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## Service registration form

This product (name): .....

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp):          Kind of work: I / S / R / D Date:	By (stamp):          Kind of work: I / S / R / D Date:	By (stamp):          Kind of work: I / S / R / D Date:
By (stamp):          Kind of work: I / S / R / D Date:	By (stamp):          Kind of work: I / S / R / D Date:	By (stamp):          Kind of work: I / S / R / D Date:
By (stamp):          Kind of work: I / S / R / D Date:	By (stamp):          Kind of work: I / S / R / D Date:	By (stamp):          Kind of work: I / S / R / D Date:
By (stamp):          Kind of work: I / S / R / D Date:	By (stamp):          Kind of work: I / S / R / D Date:	By (stamp):          Kind of work: I / S / R / D Date:
By (stamp):          Kind of work: I / S / R / D Date:	By (stamp):          Kind of work: I / S / R / D Date:	By (stamp):          Kind of work: I / S / R / D Date:

Vermeiren GROUP  
Vermeirenplein 1 / 15  
2920 Kalmthout  
BE

website: [www.vermeiren.com](http://www.vermeiren.com)