

VERMEIREN

Maintenance manual
Pedal exerciser





MAINTENANCE MANUAL

Pedal exerciser

1. Care

It is recommended to use a soapy solution for cleaning. The cleaning liquid should in any case have a pH value of at least 6. Do not use a cleaning agent which contains solvents. Observe the instructions of the cleaning liquid used. Use a damp cloth and lukewarm water for cleaning. Avoid soaking. Do not use metal brushes or any other sharp-edged or pointed cleaning utensils. In the case of damage to the varnish consult your specialist dealer. Regular care would ensure that your pedal trainer is preserved in a perfectly functional condition.

2. Disinfection

For sterilisation use only disinfectants suitable for treating varnished metal and wood surfaces. Observe the instructions of the cleaning liquid used. Use only proved processes (for scrubbing disinfection) and disinfectants included in the list of the Robert Koch Institute (information can be obtained from: www.rki.de). Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

3. Repairs / Service / Inspection

Lifetime of the pedal exerciser is influenced by its use, storage, regular maintenance, servicing and cleaning.

You should use only genuine VERMEIREN replacement parts. Repairs and renovations may only be undertaken by trained persons. Make use of the services offered by your specialist dealer. He would gladly assist you further in regard to service and repairs.

Before you resume use of the pedal exerciser, it should be serviced according to the following criteria:

- Completeness
- Adjustable resistance and pedal mechanism
- Frames (deformation, stability, connections)
- Cleaning
- **Damaged varnish (remove possible rust and repaint)**

Every 8 weeks:

- Lubricating (moving parts shall be greased)

Every 6 months or for every new user, it should be serviced according to the following criteria:

- General review
- Disinfection

4. Shipping and storage

The shipping and storage of the pedal exerciser shall be according following instructions:

- Store in a dry place (between +5°C and +41°C, +41°F and 105.8°F).
- The relative humidity of the air should be between 30% and 70%.
- Provide sufficient covering or packaging to protect the pedal exerciser from rust and foreign bodies (e.g. salt water, sea air, sand, dust).
- The pedal exerciser must be stored without being subjected to strains. (Do not put too heavy parts on the pedal exerciser, not clamping between something, ...)

5. Warranty

Excerpt from the "General Business Conditions".

(...)

5. The guarantee period for warranty claims is 24 months.

(...)

The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts.

Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

6. Disposal

When disposing of the pedal exerciser, contact your local disposal centre or return the product to your specialist dealer who, after submitting it to a hygienic procedure, will be able to send it back to the manufacturer who will dispose of and recycle it correctly, separating it into its component materials.

Packaging materials can be taken to disposal or recycling centers or to your specialist dealer.